



Installing the Anesthesia Touch software on a PC


1. Ensure Telnet is turned on in Windows Features

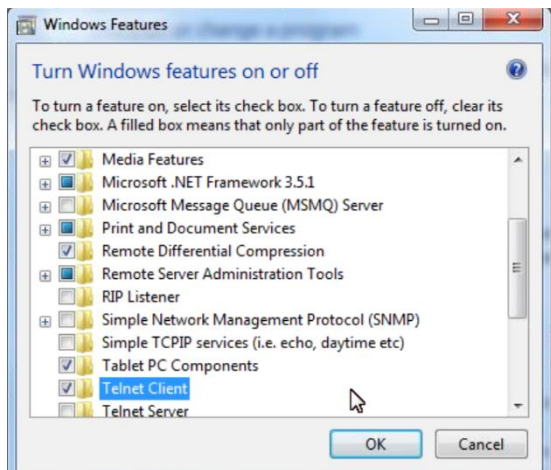


2. Download the software from build.qa.plexusis.com
3. Click Launch.
4. Click Install. This downloads the setup application.

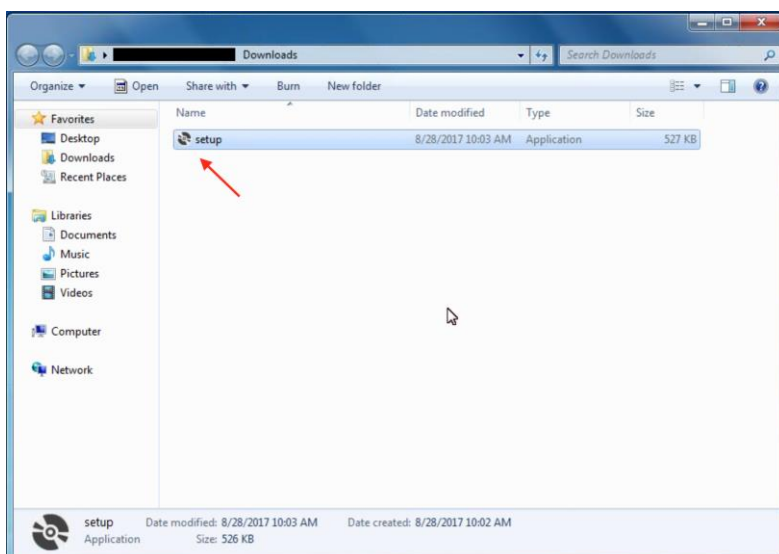


5. Double click "setup" to begin installation.

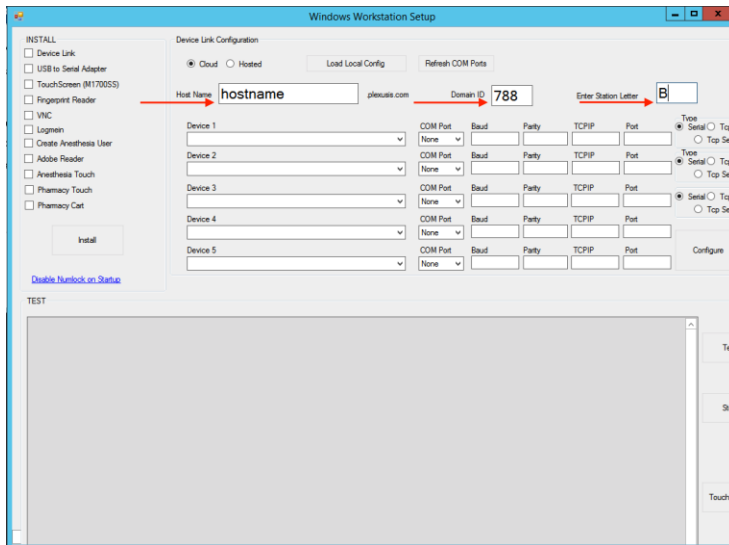


6. Enter in your Host Name, Domain ID and Station Letter (OR number).
7. Select Device Link, USB to Serial Adapter, VNC, and Anesthesia Touch under the INSTALL list. (Please see your account manager to see if the other options are necessary for your site.)

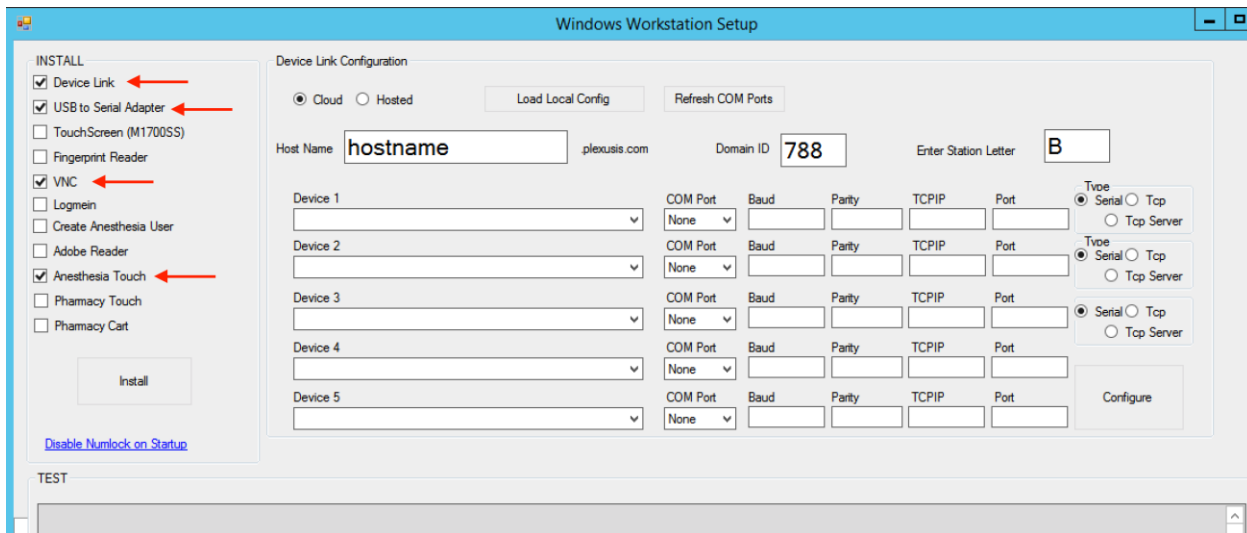

8. Click Install. Several Installation wizards will pop up. Please continue until all are installed, NOT changing any settings.
9. If .NET framework is not installed, you will be prompted to install it.


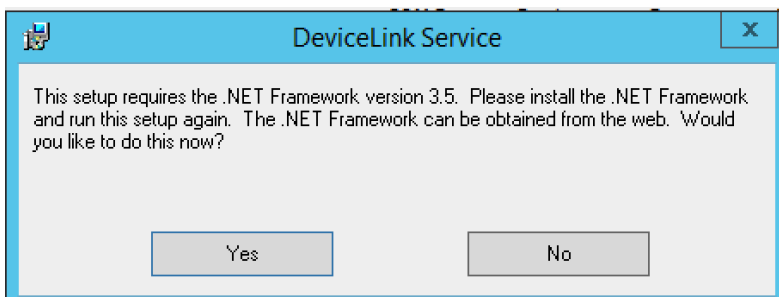


10. A browser will open to your domain Web Admin site.



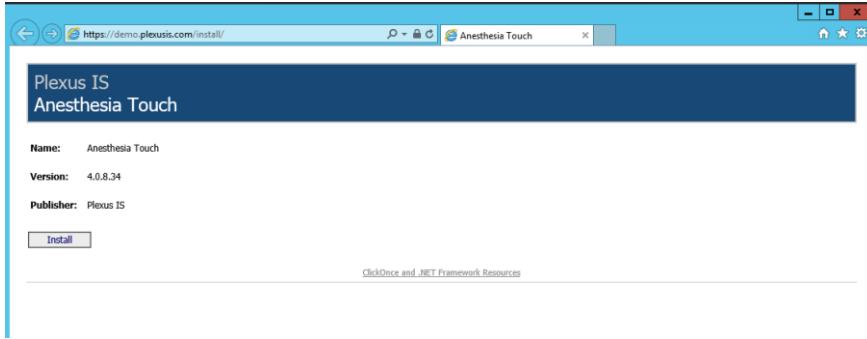


11. Click Install to begin the Anesthesia Touch installation.

12. Enter in your Host Name in the Server Options Window and click OK.

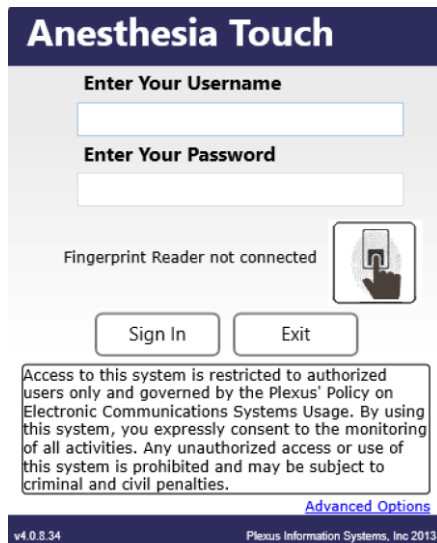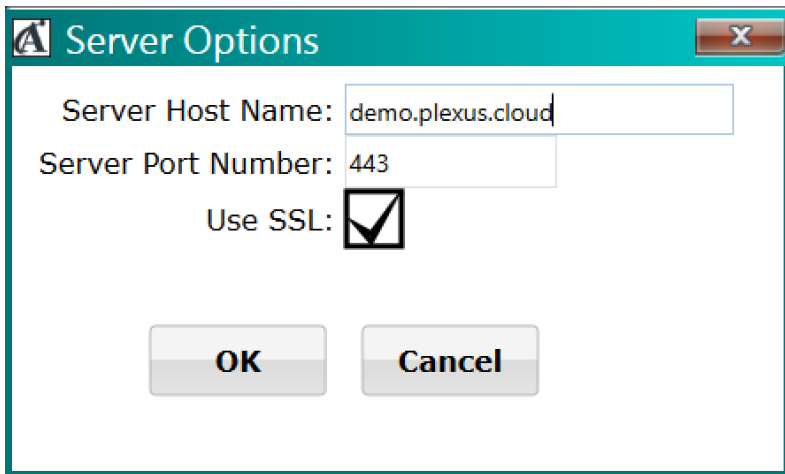


13. Log into Anesthesia Touch.

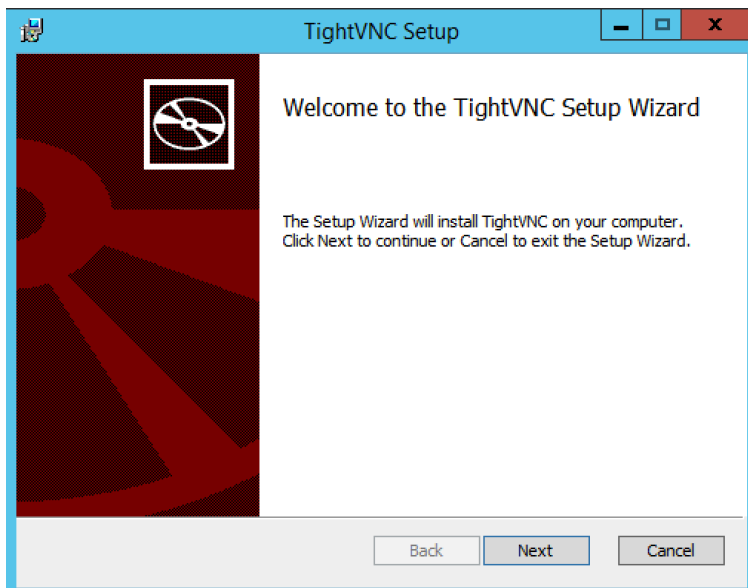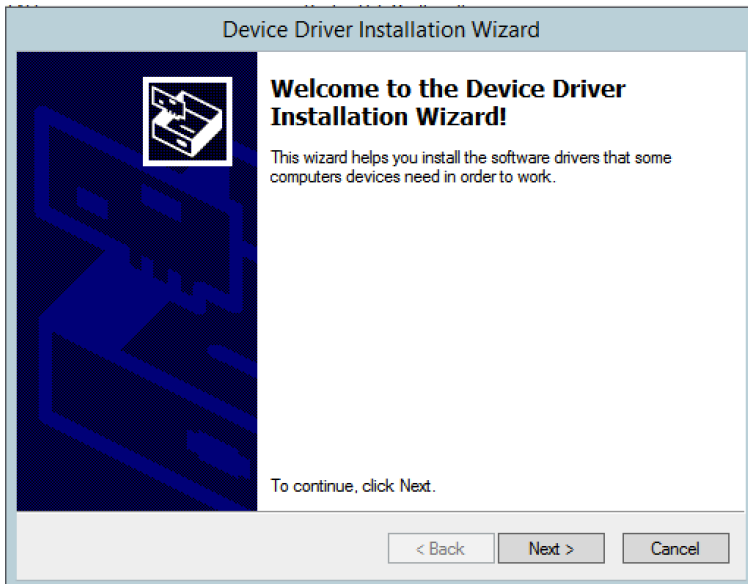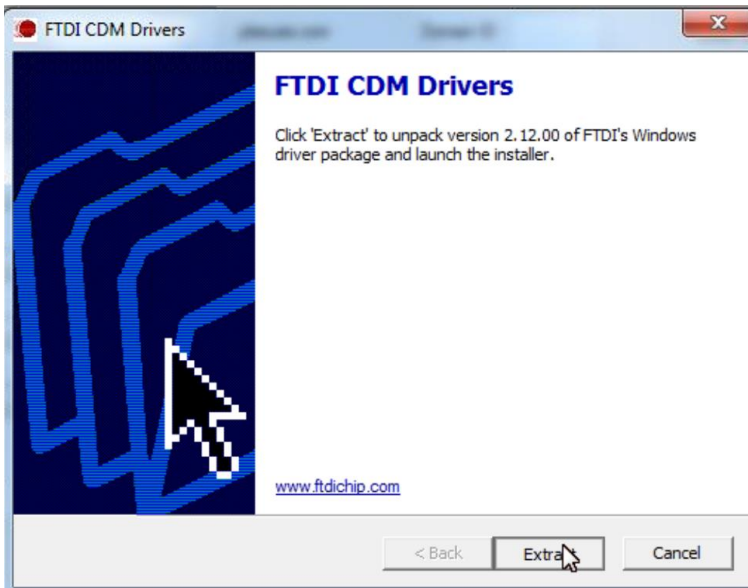

14. A shortcut to the Anesthesia Touch application will appear on your desktop.

- On the Windows Workstation Setup Screen, select the Monitors you have connected to your PC under Device 1 and Device 2 if necessary.
- Select the COM ports that are being used for those monitors. (You can use Device Manager in Windows to see which monitor is connected to which port.)

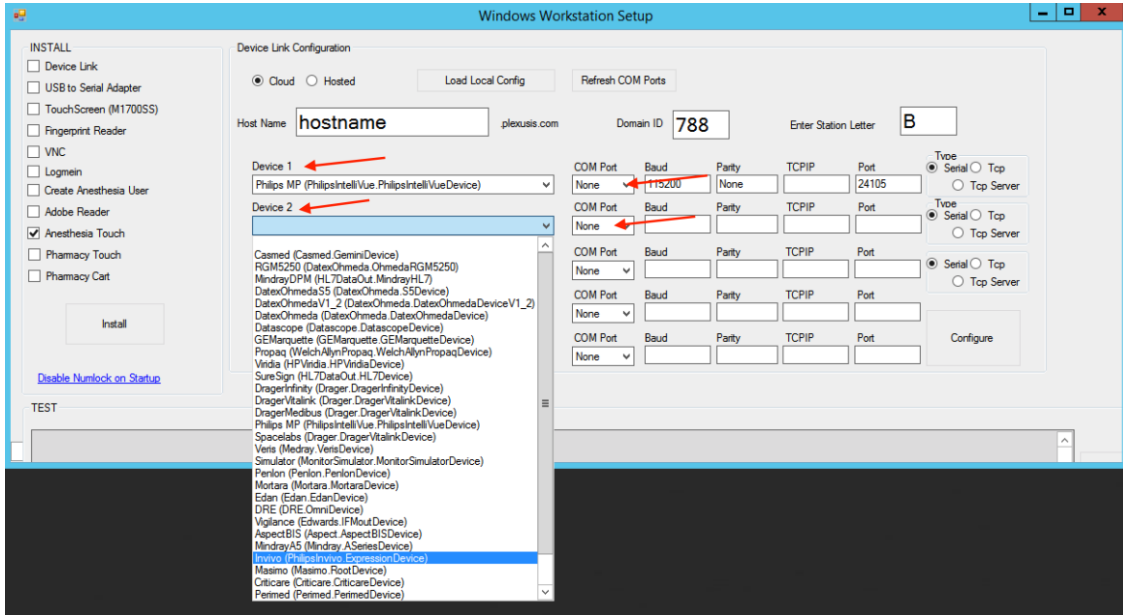

- Click Configure.
- Ensure Monitors are on and NOT in stand-by mode.
- Place your finger in the pulse/ox
- You should see data flowing in vitals area of Anesthesia Touch.