


Testing: Nexi-5

- Post installation check

- Nexi is properly mounted
- Power connection to the Nexi
- Cable connection between the Nexi and medical devices
- Medical device settings if specified
- Network cable connection if using Ethernet
- All cables and cord are properly managed

- Observe

- Power light should be solid green.
- Network light should be solid green.
- Server light should be solid green.
- Data Transmission may blink blue.

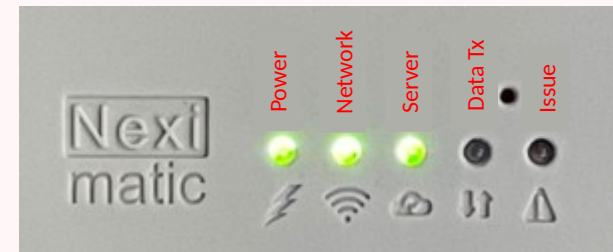

Testing: Nexi-5


- Testing

- Turn on the medical devices
- Put a Pulse Ox sensor on you
- When measurements are on the medical devices, scan the QR code on the Nexi and follow the instruction to review the live vital sign data.




Information: Nexi-5

- Starting a reboot sequence.



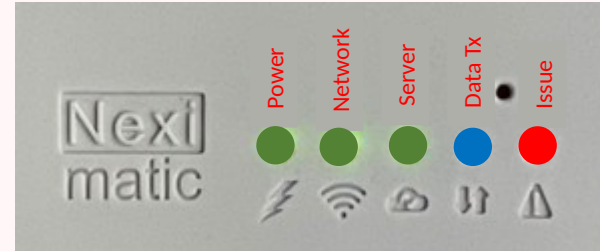


- During a reboot sequence.
- When the blue light goes off, operation begins.

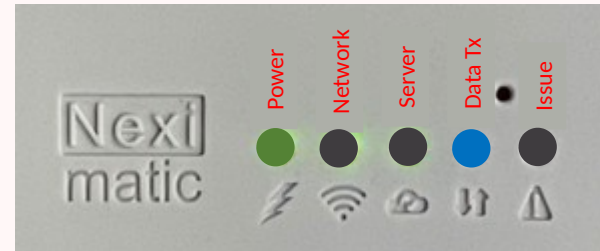

Information: Nexi-5

- Operation begins and Power light stays on.
- Regardless of WIFI or Ethernet, when there is a network connection, the Network light goes green.
- When the Nexi can reach its server, the Server light goes on green. This Server light has to be on.

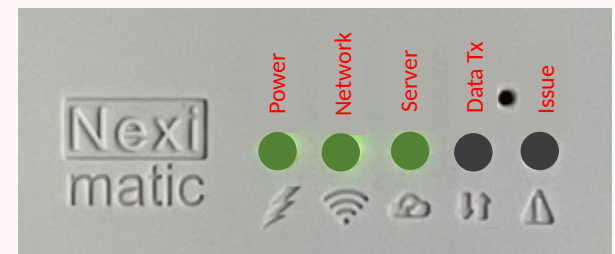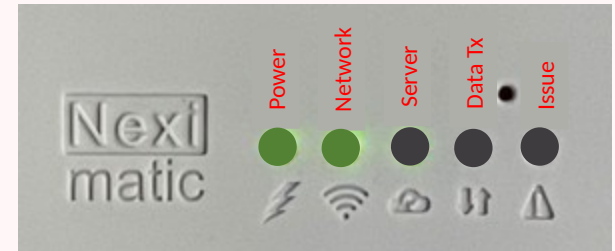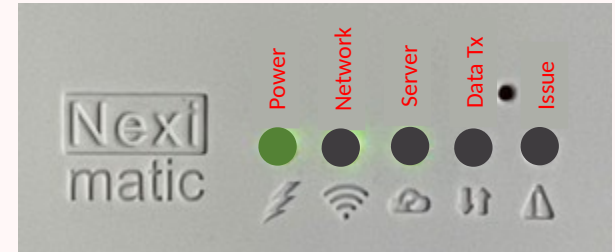

Information: Nexi-5

- When there is any issue the Issue light goes on red. The Nexi would attempt a sequence of recovery. The recovery may involve a reboot.

